SOEHNLE

Bedienungsanleitung
Operating Instructions
Mode d'emploi
Istruzioni per L'uso
Gebruiksaanwijzing
Instrucciones de manejo
Manual de instruções
Bruksanvisning
Brugsanvisning
Käyttöohjeet
Kezelési útmutató
Instrukcja obsługi
Návod k použití
Ръководство за обслужване
Kullama kılavuzu
Οδηγίες Λειτουργίας
Navodila za uporabo
Uputa za uporabu

fat

water

weight

deutsch	3
english	9
français	15
italiano	21
nederlands	27
español	33
português	39
svenska	45
dansk	51
suomi	57
magyar	63
polsku	69
česky	75
по русском языку	81
türkçe	87
ελληνικά	93
slovensko	99
hrvatski	105

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Reklamations- _____

grund _____

By choosing the Body Balance, you are making an important contribution to your health care. This electronic body monitoring scale performs the following functions:

- ▶ The Body Balance uses your personal data to **determine your body-fat/water proportion and weight.**
- ▶ 1 personal memory and 1 guest memory.

Important advice

We recommend that you consult a doctor if you are overweight and are trying to reduce your weight, or if you are underweight and are trying to put on weight. Treatment and diet should only take place in consultation with your doctor.

Recommendations on exercise programmes or weight loss diets, on the basis of the values determined, should be provided by your doctor or certified health professional. Do not try to interpret the data yourself.


SOEHNLE will not be liable for damage or loss caused by the Body Balance or for third party claims.

- ▶ This product is intended for personal home use only. It is not designed for professional use in hospitals or medical facilities.
- ▶ **Not suitable for people with electronic implants, such as pacemakers, etc.**


Preparation for use

- ① Insert batteries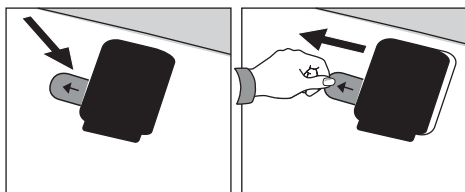


- ② Always place the scale on a level, firm surface for measurements.

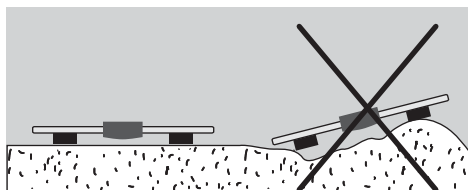- ③ Caution: danger of slipping on wet surface.

- ④ Cleaning and care: clean only with a damp cloth. Never use solvents or abrasives. Do not immerse the scale in water.

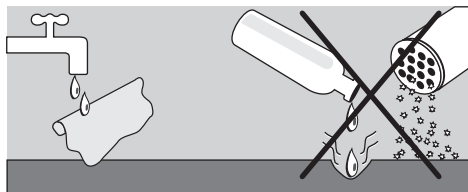
- ⑤ Country-specific settings: the scale can be switched from kg/cm to lb/in or st/in (applies to the English-language versions only).

For glass scales:


For plastic scales: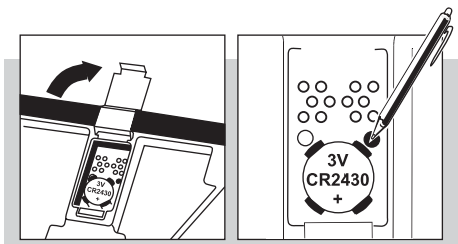


Programming


- ① Place scale on a table for programming
- ② Switch on scale (↔) and wait for **0.0** to appear in the display.
- ③ Start programming (↔).
- ④ Memory location (**P1 ... P2**) select (▼ = minus, ▲ = plus) and confirm (↔).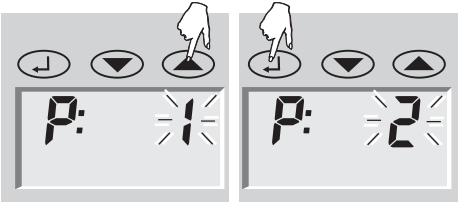


- ⑤ Enter your height (▼ = minus, ▲ = plus) and confirm (↔).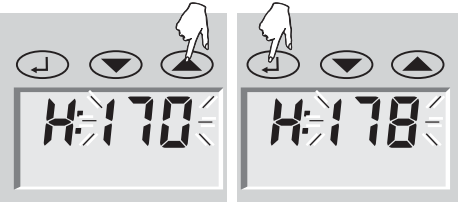

- ⑥ Enter your age (▼ = minus, ▲ = plus) and confirm (↔).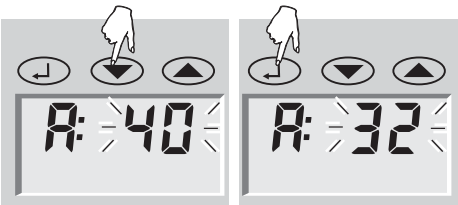

- ⑦ Input your gender (▼ = ♀ female or ▲ = ♂ male) ...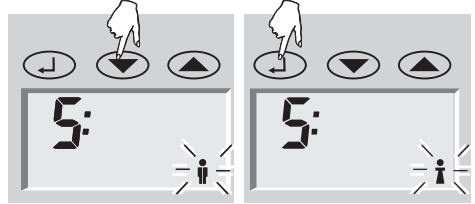

...and press for 3 sec. to confirm (↔).

Note

The scale has an energy-saving mode that switches it off automatically if no keys are pressed for approximately 1 minute.


Body monitoring

Always measure in bare feet. Personal data must be programmed first (page 11).


- ① Step on to the scale.
The display will now show your weight...

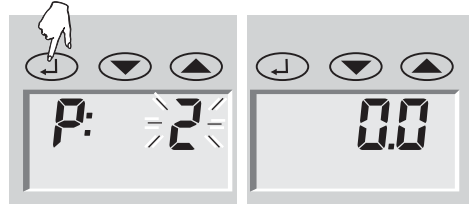
... and – while analysis is taking place (25-30 seconds) – a running bar display. Stand still until analysis is complete.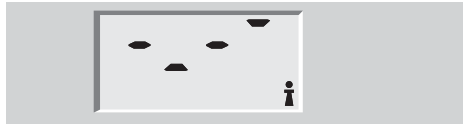

- ② Analysis takes place automatically with the data in personal memory P1.

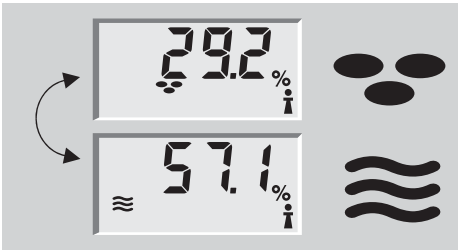
- ① Select guest memory P2, as required, before the measuring process, by pressing the  button,


Wait for 0.0 to be displayed.

- ② Step on to the scale.
As long as the analysis continues (25-30 sec.), a running segment display appears. Stand still until analysis is complete.
- ③ Indication of body weight and running segment display, ...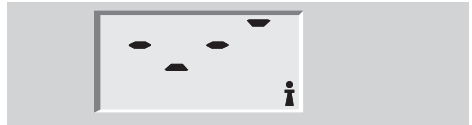

Body-water (≡) and body-fat proportions (••) are displayed alternately.

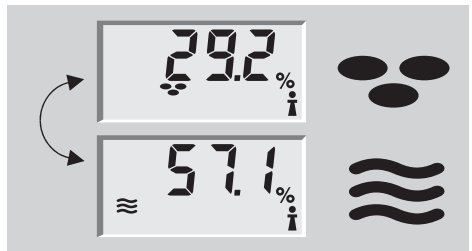- ③ Step off the scale. The scale switches off automatically when the person steps off.

Note that you will only obtain a weight reading – and no body monitoring data – if you step on the scale with your shoes or socks on!

Factors influencing measurement readings

Analysis is based on the measurement of the body's electrical resistance. Eating and drinking habits during the course of the day and individual lifestyle affect the water balance. This is noticeable by the fluctuations in the display.

In order to ensure that the results of analysis are as accurate and consistent as possible, keep the measurement conditions constant, as only in this way will you be able to observe changes over an extended period.

Other factors can affect water balance:

- ▶ After a bath, the body fat reading may be too low and the body water reading too high.
- ▶ After a meal, readings can be higher.
- ▶ Women may experience fluctuations due to the menstrual cycle.
- ▶ Due to loss of water caused by illness or after physical activity (sport). After taking exercise, wait for 6 to 8 hours before carrying out the next measurement.

Varying or implausible results can occur in the case of:

- ▶ Persons with a high temperature, symptoms of oedema or osteoporosis
- ▶ Persons undergoing dialysis treatment
- ▶ Persons taking cardiovascular medicine
- ▶ Pregnant women
- ▶ Athletes who undertake more than 10 hours of intensive training per week and have a resting pulse rate of under 60/min
- ▶ Competitive athletes and body builders
- ▶ Persons under the age of 17.

Health experts recommend the following percentages :


Men 						Women 				
Age	Body fat				Body water	Body fat				Body water
	low	normal	high	very high	normal	low	normal	high	very high	normal
10-12	< 8	8-18	18-24	>24	>64	<12	12-23	23-30	>30	>60
12-18	<8	8-18	18-24	>24	>63,5	<15	15-25	25-33	>33	>58,5
18-30	<8	8-18	18-24	>24	>62,5	<20	20-29	29-36	>36	>56
30-40	<11	11-20	20-26	>26	>61	<22	22-31	31-38	>38	>53
40-50	<13	13-22	22-28	>28	>60	<24	24-33	33-40	>40	>52
50-60	<15	15-24	24-30	>30	>59	<26	26-35	35-42	>42	>51
60+	<17	17-26	26-34	>34	>58	<28	28-37	37-47	>47	>50

Technical data



- ▶ Weighing capacity x graduation = max. 150kg x 100g
- ▶ Display resolution: Body-fat proportion: 0.1%
- ▶ Display resolution: Body-water proportion: 0,1%
- ▶ Height: 100 -250 cm
- ▶ Age: 10-99 years
- ▶ 1 personal memory and 1 guest memory.
- ▶ Size of LCD screen
- ▶ Measuring circuit: 0.035 mA RMS, 50 kHz
- ▶ Batteries: 2 x 3 V CR 2430
Batteries supplied with unit.

Symbols

- ① Batteries spent.
- ② Overload: From 150 kg.
- ③ Implausible analysis result – check individual programming
- ④ Bad foot-contact:
+ clean scale or feet or **check for sufficient skin moisture.**
- ⑤ If **skin is too dry** – moisten feet or carry out measurement after taking a bath or shower.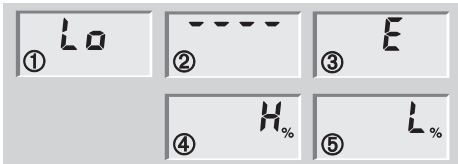

Change batteries

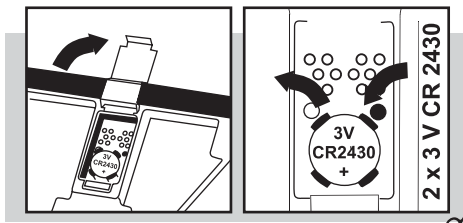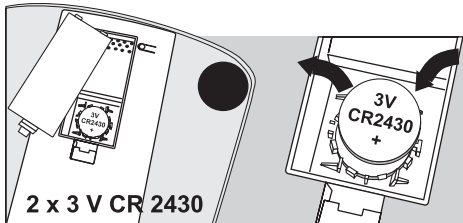Warranty

SOEHNLE guarantees that all defects due to materials or manufacturing faults will be remedied by replacement or repair, free of charge, for a period of **3 years** from the date of purchase. Please keep your purchase receipt and the guarantee card in a safe place. If you have any complaints, please return the scale to your dealer with the guarantee card and receipt.

This device is screened in accordance with the applicable EC Directive 89/336/EEC.

Note: the display value may be affected by extreme electromagnetic influences, e.g. when a radio is operated in the immediate vicinity of the device. The product can be used for its intended purpose again when the interference disappears (a reset may be required)

Disposal of spent batteries: Batteries must not be disposed of as normal household waste. Note that you are legally obliged to dispose of used batteries in a correct way. You can return spent batteries either to public collection points in your town or to any outlet selling batteries of the same kind.

Pb = Battery contains lead

Cd = Battery contains cadmium

Hg = Battery contains mercury

Guarantee card

If you have any complaints, please return the scale to your dealer with the guarantee card and your receipt.

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SOEHNLE-Waagen GmbH & Co. KG